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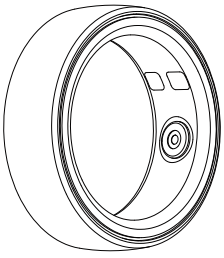
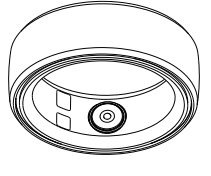
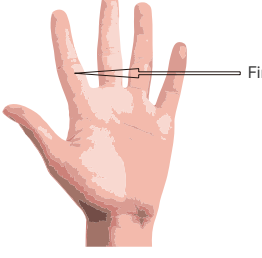
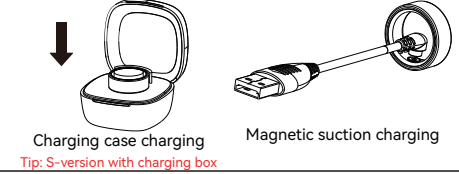
<div>Healthy Smart Ring</div> <div></div> <div>User manual</div>	<div>Product Introduction</div> <div>The data generated by this device is for reference only and does not serve as evidence for any prevention, diagnosis, or treatment of diseases.If you have any questions about your health, please consult a professional physician.</div> <div><div><div>Heart rate</div><div>Blood oxygen</div><div>Blood pressure</div><div>Calorie</div><div>Somatosensory games</div><div>Step counting</div><div>Gesture control</div><div>Waterproof level</div><div>Sleep</div></div></div>	<div>Wearing</div> <div>Wearing method: Place the ring heart rate light indicator tightly against the finger-pulp and wear it.</div> <div><div><div>Heart rate light</div><div>Finger-pulp</div></div></div>	<div>Charging and activation</div> <div>Charging activation: - When using it for the first time, please activate the device by charging it. - If the device is restored to factory settings on the APP, it needs to be reactivated.</div> <div>Ring charging: 1.Use the matching charging cable to connect the power supply and ring. 2.The red light remains on during charging, and the green light remains on after charging is completed. 3.Charging time: 60min 4.Electricity level view: APP - device page can be connected to the ring to view the electricity level.</div> <div><div><div>Charging case charging Tip: S-version with charging box</div><div>Magnetic suction charging</div></div></div>	<div>App download</div> <div>Scan the QR code on your phone or search and download the app from the app store.</div> <div></div>	<div>Pairs</div> <div>Enter the app homepage, click on the upper right corner"⊕", bring your phone closer to the device, and the app will automatically scan the device. After scanning the device, click "Connect" and follow the interface prompts to successfully add the device and enter the device control interface.</div> <div></div>	<div>Function Introduction</div> <div>1.Home page: Daily activities, exercise records, heart rate, sleep, blood oxygen, blood pressure, pressure, gesture control, somatosensory games.</div> <div>2.Sport: Multiple sports modes to choose from, can be selected different devices exercise blood pressure, blood sugar, and other functions.</div> <div>3.Device: Pair devices here, and carry out health settings, feature settings, remote photography, third-party access, firmware updates, etc.</div> <div>4.My: Fill in personal information, manage accounts, carry out permission management, set heart rate intervals, etc.</div>	<div>Specification parameters</div> <div>1.Storage capacity: It is recommended to connect to the app every day to synchronize data, depending on the type and frequency of use.</div> <div>2.Product Weight: About 6.8g</div> <div>3.Battery capacity: 18mAh</div> <div>4.Battery life: 6.5 days</div> <div>5.Bluetooth version: 5.1</div> <div>6.Bluetooth range: 3-5m</div> <div>7.Waterproof rating: IP68</div>	<div>Frequently Asked Questions and Disclaimer</div> <div>Frequently Asked Questions</div> <div>1.Unable to connect to Bluetooth Please ensure that the Bluetooth and location permissions of the app are turned on, and keep the ring fully charged.</div> <div>2.Inaccurate data Please wear the ring correctly according to instructions in manual and ensure that the sensor inside ring is in close contact with finger-pulp.</div> <div>3.Unable to find the ring The ring can only be connected to one device, if you can't find it, please confirm whether the ring is blue with your phone.If it is connected, please disconnect it and use the APP to reconnect Ring.</div> <div>Disclaimer</div> <div>1.Normal wear and damage, including scratches and dents.</div> <div>2.Consumable components included in the product, such as batteries.</div> <div>3.Unauthorized changes or repair of products resulting in damage.</div> <div>4.Damage caused by force majeure such as fire, lightning, floods, etc.</div> <div>5.Damage caused by failure to use the product according to the instructions.</div>
---	---	--	--	---	---	--	--	---

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